


# Customize the Session window

What do you want to do?

- [Dock/Undock Event log or Time Line](#)
- [Dock/Undock panes and tabs](#)
- [Display a selection of events](#)
- [Collapse all subjects](#)

## Dock/Undock Event log or Time Line

You can undock and move the Event log and Time Line window, for example to display them on a separate monitor. To dock the windows again, drag the window to the main Viso window. Docking buttons appear. Move the window to one of these docking items. A blue rectangle now shows where the window will be positioned. Release the mouse button.

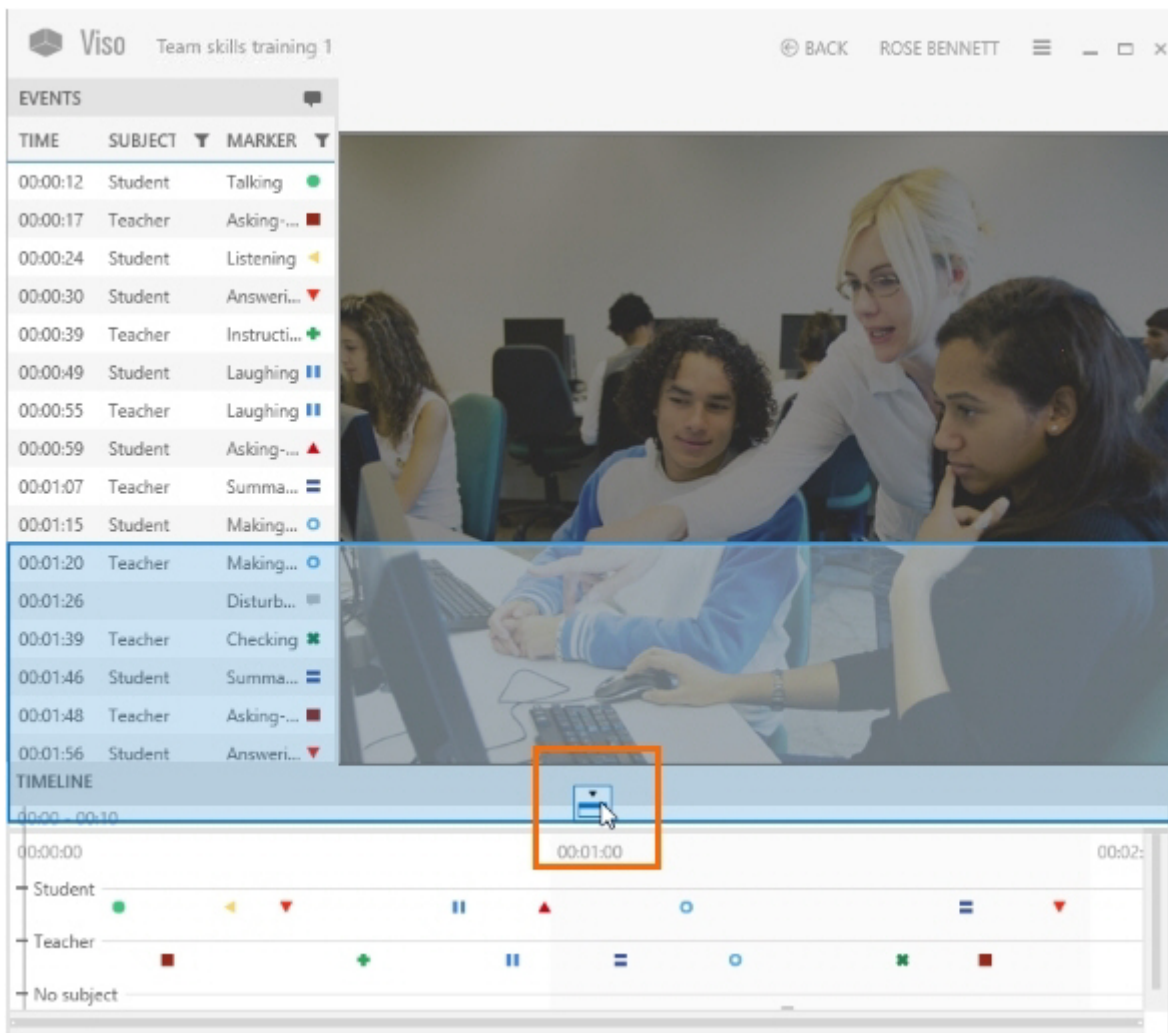


## Dock/Undock panes and tabs

You can undock the pane with the **Events**, **Codes** and **Participant** tabs, and the individual tabs. It can, for example, be useful to move the **Codes** tab to a separate monitor.


Docking/undocking of panes and tabs applies to Viso Viewer and is implemented for all users.

## To undock a pane with its tabs

Click the title and drag it to a new position.

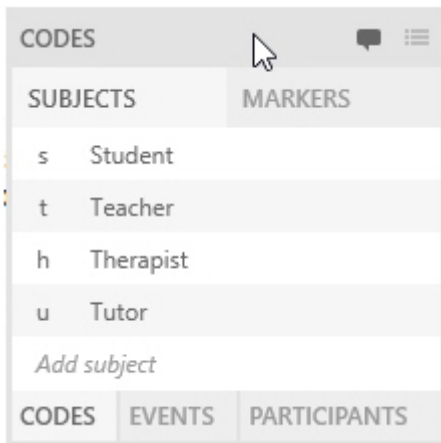


To dock a pane again, drag it to the main Viso window. Docking buttons appear. Move the pane to one of these docking items. A blue rectangle now shows where the pane will be positioned. Release the mouse button.

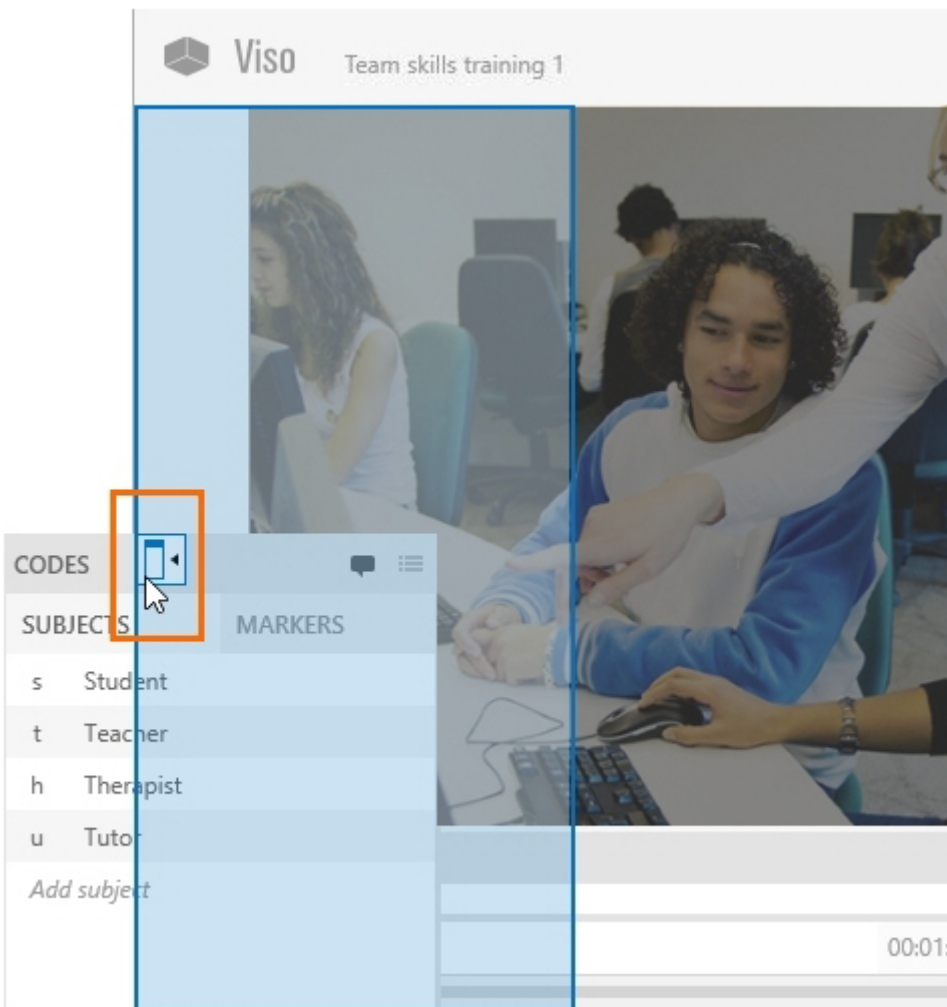


## To undock a separate tab

Click the tab at the bottom of the pane and drag it to a new position.

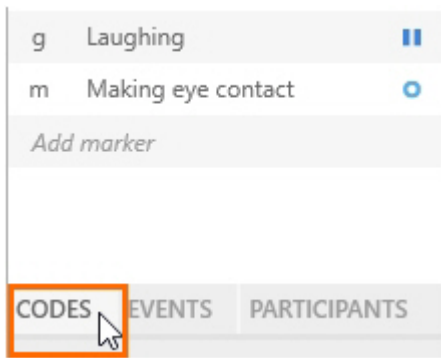


To dock the tab again, drag it to the main Viso window. Two types of docking buttons appear.

If you release the mouse button while it is positioned on the docking button shown in the figure below, it is redocked in the pane.

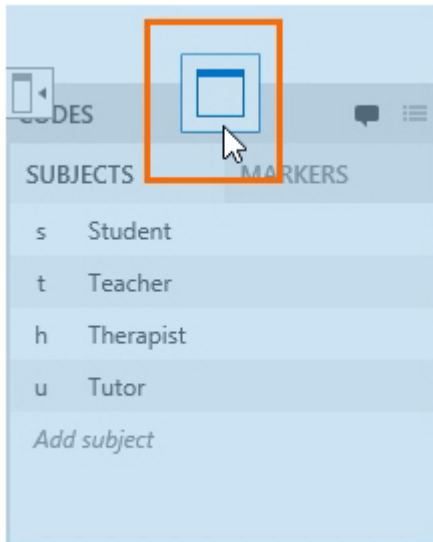

If you release the mouse button while it is positioned on the docking button shown in the figure below, the separate tab is docked in the Viso window.

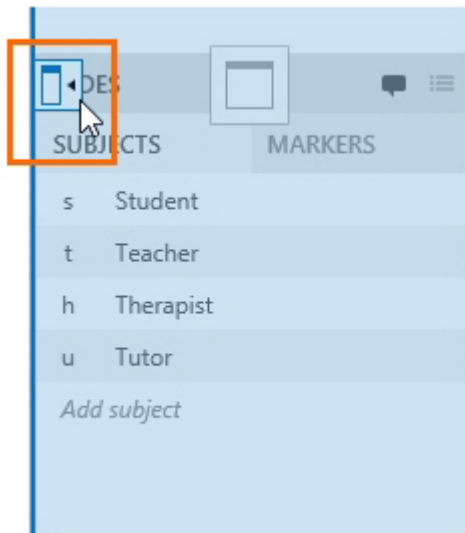


## Display a selection of events

### aim

To select a subset of events in the Event log and, optionally, in the Timeline. This can be useful to, for example, quickly display all behaviors of one subject, or a specific behavior of all subjects.

## procedure


1. Open the **Events** pane.
2. Click the filter button next to **Subject** or **Marker**.

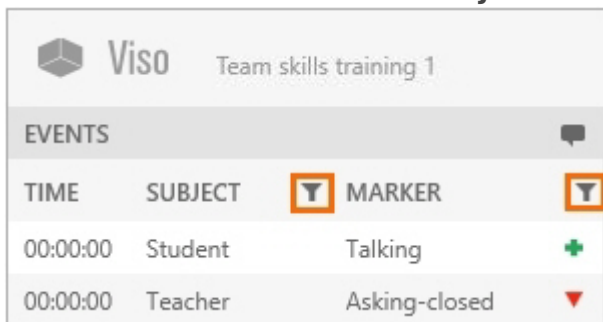

The screenshot shows the Viso interface with the title 'Team skills training 1'. The 'EVENTS' pane is active, displaying a table with columns: TIME, SUBJECT, and MARKER. The filter buttons (funnel icons) next to the SUBJECT and MARKER headers are highlighted with orange boxes. The table contains two rows of data:

TIME	SUBJECT	MARKER
00:00:00	Student	Talking
00:00:00	Teacher	Asking-closed


3. Specify the selection. The result is shown in the Event log.

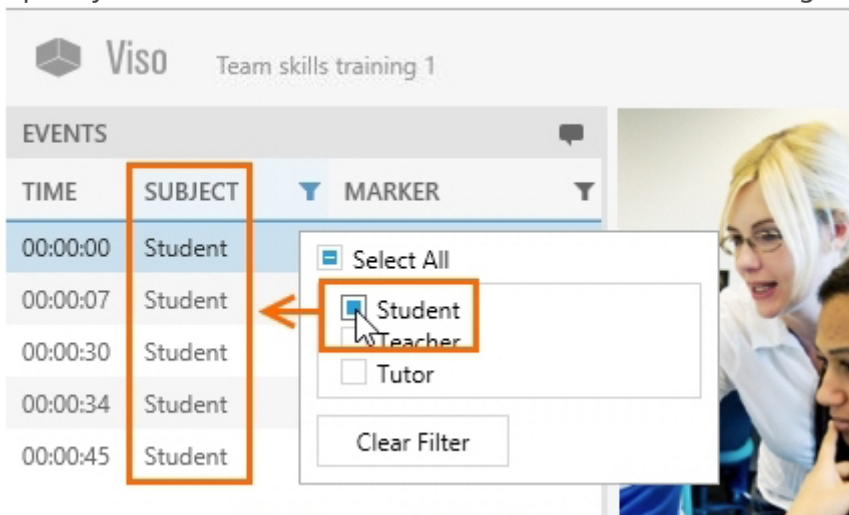

The screenshot shows the Viso interface with the 'EVENTS' pane. The 'SUBJECT' column header is highlighted with an orange box. A dropdown menu is open, showing a list of subjects: 'Select All', 'Student', 'Teacher', and 'Tutor'. The 'Student' option is selected and highlighted with an orange box. A blue arrow points from the 'Student' option in the dropdown to the 'Student' entries in the table below. The table shows five rows, all with 'Student' as the subject:

TIME	SUBJECT	MARKER
00:00:00	Student	
00:00:07	Student	
00:00:30	Student	
00:00:34	Student	
00:00:45	Student	


The filter is also applied in the Timeline.

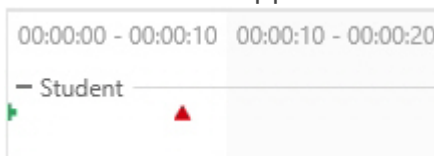

## To disable the filter in the Timeline

To see in what context the selected events took place, you may want to display all events in the Timeline, instead of only the ones specified in the Event log filter. To do so:

1. Right-click the Timeline.
2. Select **Show all events**.

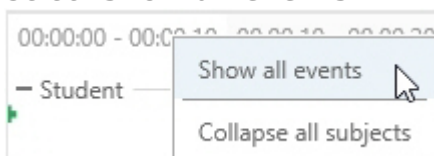

**NOTE** To restore the filter in the Timeline, right-click it and select **Show filtered events only**.

## Collapse all subjects

aim

To quickly collapse all subjects in the Timeline, so that no markers are shown. You can then expand the subjects that you are interested in to view only their markers.

00:00:00 - 00:00:10	00:00:10 - 00:00:20
+ Student	
+ Teacher	
+ Tutor	

## procedure

Right-click the Timeline and select **Collapse all subjects**.

**NOTE** To expand all collapsed subject again, right-click the Timeline and select **Expand all subjects**.

---

Revision #4

Created 15 January 2024 11:38:50

Updated 11 March 2024 14:26:38