

Keyboard shortcuts

Why use keyboard shortcuts

Keyboard shortcuts allow you to use Viso without taking your hands off the keyboard. Using keyboard shortcuts can also help in the prevention of repetitive strain injury. Keyboard shortcuts are only available if the corresponding menu or window is active. Viso has keyboard shortcuts for:

- [General functionality](#)
- [The Sessions tab](#)
- [The Codes tab \(Annotation Module\)](#)
- [Record sessions](#)
- [Review sessions](#)
- [The menu](#)

When you change the Viso terminology, the shortcut keys remain unchanged. For example, if you change ‘Locations’ into ‘Rooms’ the shortcut key to open the Rooms tab is still Alt+ L.

General functionality

| | | |
|----------|---------------------------|-----|
| Ctrl + L | Login / logout | |
| F1 | Open Viso Help | |
| Alt + F4 | Exit Viso | |
| Alt + D | Open Dashboard | |
| Alt + L | Open Locations tab | L |
| Alt + S | Open Sessions tab | |
| Alt + H | Open Scheduler tab | L |
| Alt + C | Open Codes tab | L,A |
| Alt + G | Open Groups window | L |

L – available for users with a role that allows the action.
A – only available with Annotation Module.
You can see the shortcut keys by hovering with your mouse over the menu items.

The Sessions tab

| | | |
|-----|-----------------|---|
| Del | Delete sessions | L |
| F2 | Rename session | L |

L – available for users with a role that allows the action.

Please note that you need to select a session before you can use the shortcut keys. You can select a session by clicking the downwards or upwards arrow keys on your keyboard.

The Codes tab (Annotation Module)

| | | |
|-----|--------------------|---|
| Del | Delete codes lists | L |
|-----|--------------------|---|

L – available for users with a role that allows the action.

Please note that you need to select a codes list before you can use the shortcut key.

Record sessions

| | | |
|--------------------|--|-----|
| Backspace | Exit session | L |
| Ctrl + Shift + R | Start / Stop Recording | L |
| Ctrl + [n] | Enable / disable PTZ in video [n] | L |
| Ctrl + Shift + [| Save 1st PTZ preset | L |
| Ctrl + Shift +] | Save 2nd PTZ preset | L |
| Ctrl + [| Return to 1st saved PTZ preset | L |
| Ctrl +] | Return to 2nd saved PTZ preset | L |
| Numpad 1 | Move camera left-down | L |
| Numpad 2 | Move camera down | L |
| Numpad 3 | Move camera right-down | L |
| Numpad 4 | Move camera left | L |
| Numpad 6 | Move camera right | L |
| Numpad 7 | Move camera left-up | L |
| Numpad 8 | Move camera up | L |
| Numpad 9 | Move camera right-up | L |
| Numpad + | Zoom in | L |
| Numpad - | Zoom out | L |
| Alt + [n] | Enable Push-to-talk in camera [n] | L |
| Ctrl + Shift + [n] | Only show video [n]. Press it again or press Esc to show all previously visible videos. | L |
| Esc | Show all previously visible videos (after double-clicking a video, or pressing Ctrl + Shift + [n]) | L |
| Ctrl + O | Select codes list | L,A |
| Ctrl + R | Add remark | L,A |
| Del | Delete marker, remark | L,A |

| | | |
|--------------------|---|-----|
| Ctrl + Q | During annotating, switch between Subjects and Markers tab. | L,A |
| Page up | Open next location in carousel control | L,M |
| Page down | Open previous location in carousel control | L,M |
| Alt + C | Switch to Codes tab | L,A |
| Alt + E | Switch to Events tab | L,A |
| Alt + P | Switch to Participants tab | L |
| Shift + [n] | Mute/unmute video [n] | |
| Shift + M | mute/unmute all videos at once | |

L – available for users with a role that allows the action.

A – only available with Annotation Module.

M - the Setup account selected multiple-location recording.

Control camera with CH products RS Desktop joystick

Viso supports the CH products RS Desktop joystick for Pan, Tilt, and Zoom control of IP cameras. The joystick and buttons are predefined in the following way:

| | | |
|-----------------|-----------------------------------|---|
| Joystick | Pan, Tilt, Zoom | |
| 1 | Enable / disable PTZ in video [1] | L |
| 2 | Enable / disable PTZ in video [2] | L |
| 3 | Enable / disable PTZ in video [3] | L |
| 4 | Enable / disable PTZ in video [4] | L |
| 5 | Return to Preset 1 | L |
| 6 | Return to Preset 2 | L |

L – available for users with a role that allows the action.

Review sessions

| | | |
|---------------------------|--|-----|
| Spacebar | Play / pause | |
| Ctrl + Backspace | Quick review | |
| Backspace | Exit session | |
| Alt + B | Exit session | |
| Ctrl + Shift + [n] | Only show video [n]. Press it again or press Esc to show all previously visible videos. | |
| Esc | Show all previously visible videos (after double-clicking a video, or pressing Ctrl + Shift + [n]) | |
| Ctrl + O | Select codes list | L,A |

| | | |
|--------------------|--------------------------------------|-----|
| Ctrl + R | Add remark | L,A |
| Del | Delete marker, remark | L,A |
| F2 | Edit remark | L,A |
| F7 | Create Highlight (only in event log) | |
| Ctrl + S | Save changes to the event log | L |
| Ctrl + ? | Skip to next event | |
| Ctrl + ? | Skip to previous event | |
| Alt + C | Switch to Codes tab | L,A |
| Alt + E | Switch to Events tab | L,A |
| Alt + P | Switch to Participants tab | L |
| Shift + [n] | Mute/unmute video [n] | |
| Shift + M | mute/unmute all videos at once | |

L – available for users with a role that allows the action.

A – only available with Annotation Module

The menu

| | | |
|----------------|--|---|
| Alt + F | Open menu | |
| V | Open Viso Help | |
| T | Open Settings window | |
| U | Open User Management window | L |
| Alt + G | Add Groups | L |
| E | Open Export Audit Log window | L |
| O | Allows you to select Viso Home Page or Customer Support Center | |
| D | Open the Upgrade window | L |
| A | Open the About Viso window | |
| O + P | Open Viso Home page | |
| O + C | Open MyNoldus sign-in webpage | |

L – available for users with a role that allows the action.

When you open the menu, the shortcut keys are shown as underscores for each menu item.

Revision #4

Created 15 January 2024 11:38:52

Updated 11 March 2024 14:37:24 by Niek Wilmink