

# Select multiple locations to record separate sessions for each location

## aim


To select multiple locations for simultaneous session recording. Each location results in a separate session.

Prerequisites

- The setup account enabled multiple-location recording.
- Your user role allows session recording (Technician, Administrator, or Trainer).

## procedure


1. In the **Locations** tab, select the locations.
2. Click the **Separate sessions** button on the toolbar.

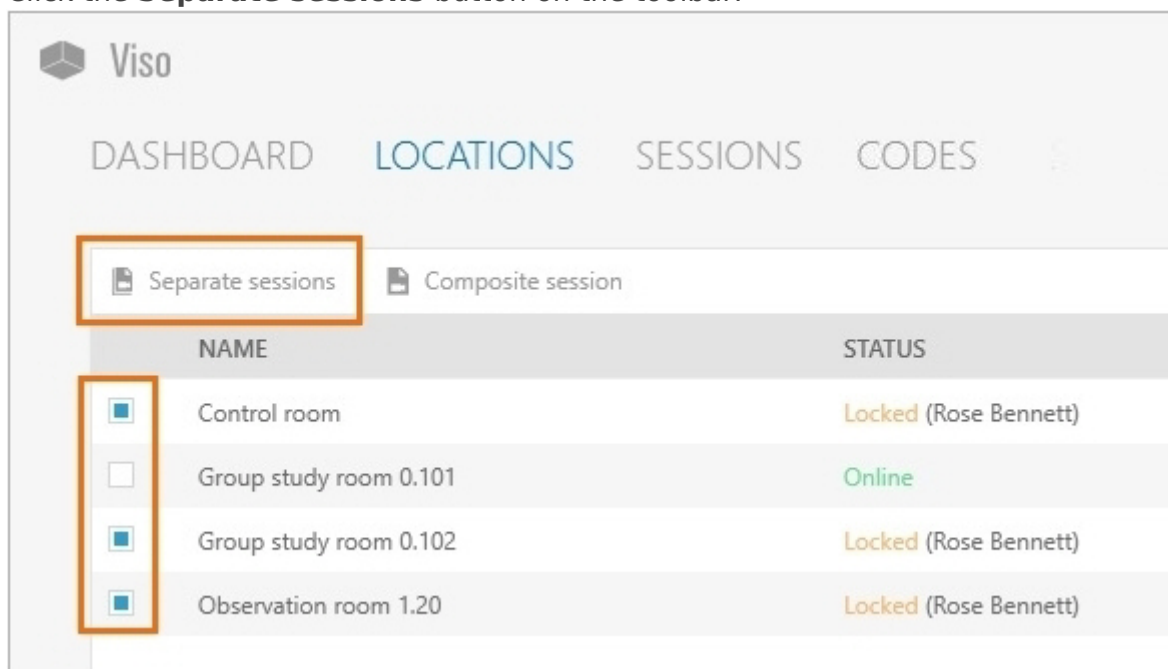


3. A preview of the cameras in the location that you selected first appears. The toolbar displays the carousel control.

## The carousel control


The location you selected first is displayed in the middle of the carousel control.

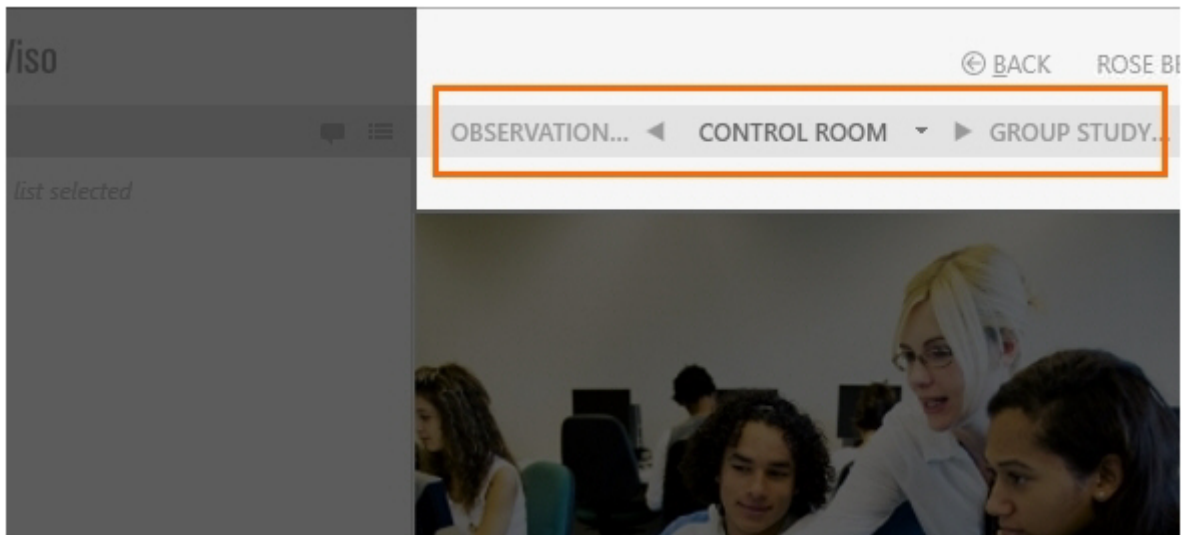

The location displayed on the right is the location you selected second.

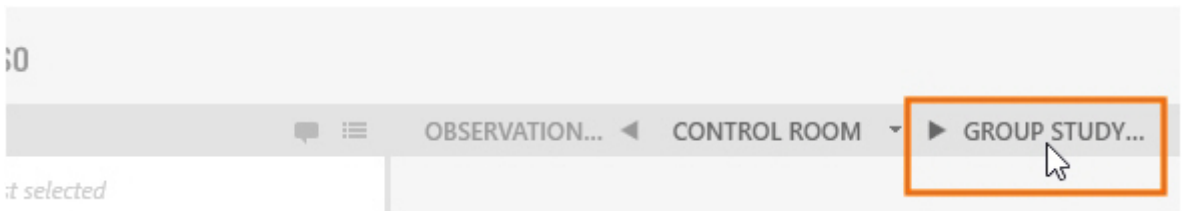


The location displayed on the left is the location you selected last.

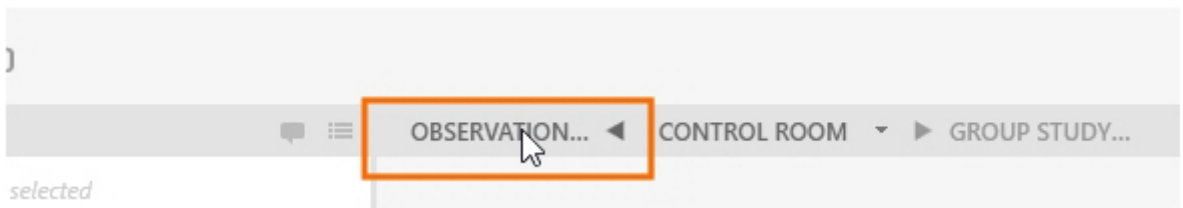

## To open locations with the carousel control

Do one of the following:

- Press the carousel control buttons to browse through the locations.
- Press the **Page up** and **Page down** button on your keyboard.
- Click the down arrow next to the location in the middle and select another location from the list.

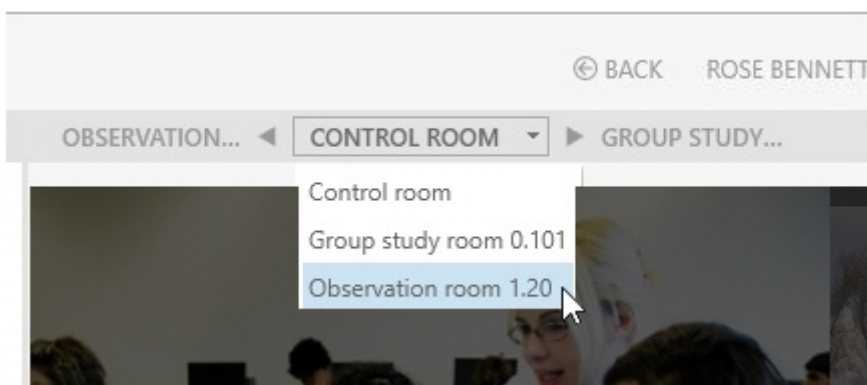

## To set up the locations

See the third chapter of this Reference ([Prepare session recording](#)) for the procedure. Follow the procedure for each location.

**NOTE** If you selected only two locations, the second is displayed on the left as well as on the right in the carousel control.

**NOTE** The **Separate Sessions** button is enabled once you selected multiple locations. Separate session recording is supported with a maximum number of eight locations with two cameras each.

---

Revision #3

Created 15 January 2024 11:38:47

Updated 12 March 2024 09:16:45